

MAJOR CHAIR COMPONENTS

HSP Integra 2001 Dental Chair - Plastic Covers

REMOVE HEADREST PLASTIC COVER:

1. Remove 4 screws to remove plastic cover.
2. Remove plastic cover.

TO INSTALL NEW HEADREST PLASTIC COVER:

1. Remove 4 screws to remove plastic cover.
2. Remove plastic cover.

TO INSTALL NEW CHAIR BACK PLASTICE COVER:

1. Replace new plastic cover.
2. Replace 4 screws to secure new plastic cover.

TO REMOVE SEAT SIDE PLASTIC COVERS (L&R):

1. Remove 2 screws inside cover, to remove plastic cover.
2. Remove plastic cover.

TO INSTALL NEW SEAT SIDE PLASTC COVERS (L&R)

1. Replace new plastic cover.
2. Replace 2 screws inside cover, to secure new plastic cover.

TO REMOVE LEGREST PLASTIC COVER:

1. Remove 4 screws on bottom side plastic cover.
2. Remove plastic cover.

TO INSTALL NEW LEGREST PLASTIC COVER:

1. Replace new plastic cover.
2. Replace 4 screws to secure new plastic cover.

TO REMOVE BASE TOP SIDE PLASTIC COVERS (L&R):

1. Remove 2 screws on top of cover, to remove plastic cover.
2. Remove plastic cover.

TO INSTALL BASE TOP SIDE PLASTIC COVERS (L&R):


1. Replace new plastic cover.
2. Replace 4 screws to secure new plastic cover.

TO REMOVE BASE CANTILEVER PLASTIC COVERS (TOP & BOTTOM):

1. Remove 2 screw on each side of top and bottom covers.
2. Remove covers.

TO INSTALL BASE CANTILEVER PLASTIC COVERS (TOP & BOTTOM):

1. Replace covers.

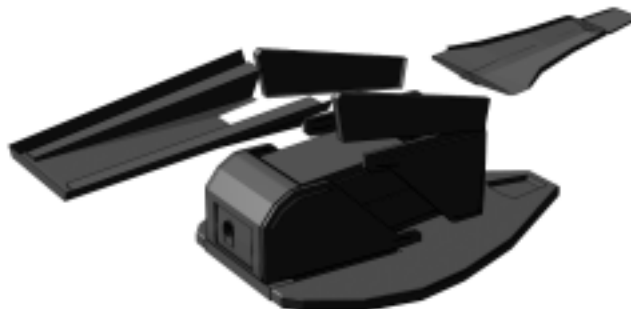

TO REMOVE BASE FRONT PLASTIC COVER (LEFT, CENTER, RIGHT)

1. Remove 1 screw on top of each side of 3 part base front housing plastic cover.
2. Remove plastic cover.

TO INSTALL BASE FRONT PLASTIC COVER (LEFT, CENTER, RIGHT)

1. Replace plastic cover.
2. Replace 1 screw on top of each side of 3 part base front housing plastic cover.

REMOVE BASE SERVICE DOOR PLASTIC COVER:

1. Remove 4 screws to remove plastic cover.
2. Remove plastic cover

TO INSTALL NEW BASE SERVICE DOOR PLASTIC COVER:

1. Replace new plastic cover.
2. Replace 4 screws on new plastic cover.

REMOVE BASE PLATE PLASTIC COVER:

1. Remove 3 screws to remove plastic cover.
2. Remove plastic cover.

TO INSTALL BASE PLATE PLASTIC COVER:

1. Replace new plastic cover.
2. Replace 3 screws to secure new plastic cover.

MAJOR CHAIR COMPONENTS

HSP Integra 2001 Dental Chair - Upholstery

REMOVE SEAT CUSHION UPHOLSTERY:

1. Remove 4 screws to remove plastic cover under toe board.
2. Remove 2 bolts under toe board of seat cushion.
3. Remove 2 bolts under the bottom of the seat cushion.
4. Remove seat cushion upholstery.

TO INSTALL NEW SEAT CUSHION UPHOLSTERY

1. Replace new seat cushion upholstery.
2. Replace 2 bolts under toe board of seat cushion.
3. Replace 2 bolts under the bottom of the seat cushion.
4. Replace plastic cover with 4 screws under toe board.

TO REMOVE CHAIR BACK & SLING UPHOLSTERY:

1. Remove chair back plastic cover (4 screws).
2. Remove 4 bolts in chair backrest cushion.
3. Remove chair backrest upholstery.

TO INSTALL NEW CHAIR BACK & SLING UPHOLSTERY:

1. Replace new backrest upholstery.
2. Replace 4 bolts in chair backrest cushion.
3. Replace chair back cover (4 screws).

TO REMOVE ARM REST UPHOLSTERY:

1. Remove 6 screws in bottom of arm rests.
2. Remove armrest upholstery.

TO INSTALL NEW ARM REST UPHOLSTERY:


1. Replace armrest upholstery.
2. Replace 6 screws in bottom of armrests

TO REMOVE HEADREST UPHOLSTERY:

1. Remove 4 screws on back of plastic cover of headrest.
2. Remove 4 bolts in headrest cushion.
3. Remove headrest upholstery.

TO INSTALL NEW HEADREST UPHOLSTERY:

1. Replace headrest upholstery.
2. Replace 4 bolts in headrest cushion.
3. Replace plastic cover on back of headrest with 4 screws.

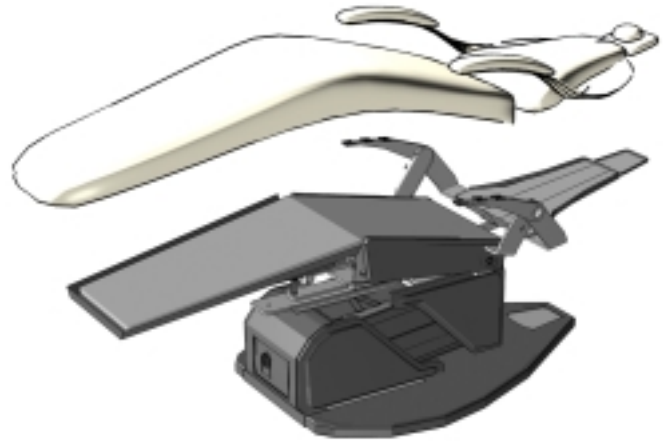

GENERAL INFORMATION

HSP Integra 2001 Dental Chair Cleaning & Disinfecting Guide



Taking Care of the Chair's Vinyl Upholstery

The following instructions are suggested for cleaning and disinfecting:

Care and Cleaning Guide

<u>Staining Agent</u>	<u>Cleaning Procedure</u>
Eye Shadow/Mascara	Step I
Grease	Step I
Crayon	Step I
Black Felt Tip Pen	Step I
Chocolate	Step I
Suntan Lotion	Step I
Red Lipstick	Step 2
Blood, Urine, Feces	Step 3
Oil Base Paint, Tar, Asphalt	Step 3
Other Tough Stains	Step 4 or 5

Step 1 - Remove excess spill with damp cloth. Clean with 1: 1 mix of Ivory liquid and water. Rinse with clean water and dry.

Step 2 - Use straight application of concentrated cleaners such as Formula 409 or Fantastik Spray Cleaner. Then wipe with clean cloth.

Step 3 - Use a 1: 1 mix of ammonia and water or a 1:4 mix of bleach and water. Rinse with clean water and dry.

Step 4 - Use straight application of naphtha (lighter fluid). Rinse thoroughly with clean water and pat surface dry.

Step 5 - Use 1: 1 mix of isopropyl alcohol and water. If stain persists, use straight alcohol. Rinse thoroughly with clean water and pat surface dry.

Note: For cleaning that requires steps 4 or 5 - use a soft cotton cloth saturated with the cleaning material, rub the stain in circles 10 times. Pat dry with another soft cotton cloth and check results.

Disinfecting Procedural Guide

After each patient use, clean and disinfect the dental chair. It is recommended that an EPA approved disinfectant always be used to properly disinfect the dental chair. The disinfectant procedure is to spray disinfectant solution on all chair surfaces that may come in contact with the patient.

HSP recommends the **Virabol** brand EPA approved hospital disinfectant that will not stain or discolor the vinyl upholstery. Other types of disinfectant formulas may create discoloration of the upholstery materials.

NOTE: These are recommended or suggested methods of cleaning and disinfecting, but the manufacturer is not responsible for damage incurred while cleaning or disinfecting.

ALWAYS TRY THE CLEANING AND DISINFECTANT METHOD
IN A HIDDEN AREA FIRST TO CONVINCCE YOURSELF OF THE RESULTS.