

## GENERAL INFORMATION

### HSP Integra 2001 Dental Chair - Trouble Shooting Guide

#### **A. Chair not working - Check the following**

1. Is the power to the chair on? Check the building circuit breaker.
2. Is the chair plugged into the electrical outlet?
3. Check the fuses located on the circuit board in the toe of the dental chair. One fuse is located on the upper left of the circuit board and another is located on the lower left on the circuit board.
4. If a fuse is blown, replace it with the proper amp of fuse. Upper fuse is the primary 1/16 amp and the lower fuse is the motor fuse, 6-amp fuse. Refer to page 10.

#### **B. Chair lost the programming, which you have programmed**

For this problem, reprogram chair (gray and black buttons). Refer to page 6.

#### **Review on programming:**

Position chair base and back to the desired patient position and then press the **gray** button down and hold it until the audible beep is heard, then quickly release the button. Repeat the procedure for the **black** button.

#### **General Maintenance of Chair:**

- **Weekly:** Clean the chair, including upholstery and painted items with warm water and a mild soap. Refer to page 19.
- **Every Six Months: Lubricate** both base motor and back motors' spiral screw tubes with Lithium Grease.

If other problems develop, contact Health Science Products at: 205-251-0500